





大便样本收集指引（大便隐血测试）： Stool Sample Collection Instructions (FIT Test):

1. 扭开并拉出绿色樽盖。
Open the cap of the bottle by twisting & lifting.

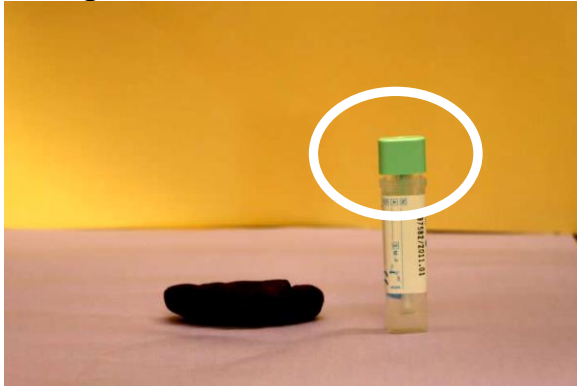

2. 确认试管棒底部的坑纹向下。
Ensure that the grooved side is pointing toward the stool.

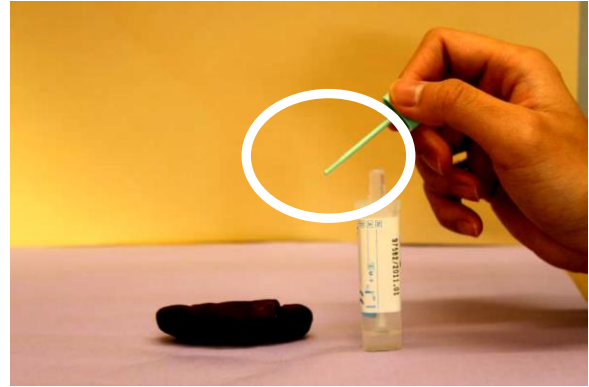

3. 用试管棒底部坑纹插入大便 6 个不同地方，以收集样本。
Scrap the stick on the stool in 6 different areas until the grooved part is covered by the samples.



4. 用力将试管棒插回樽内，并扭动关上。于空白位置填上收集大便日期和时间。
Insert the stick and twist the cap closed. Write down the date and time on the surface of the bottle.

