





大便樣本收集指引（大便隱血測試）： Stool Sample Collection Instructions (FIT Test):

1. 扭開並拉出綠色樽蓋。
Open the cap of the bottle by twisting & lifting.

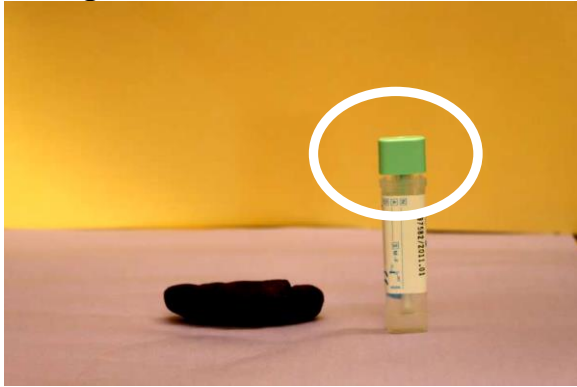

2. 確認試管棒底部的坑紋向下。
Ensure that the grooved side is pointing toward the stool.

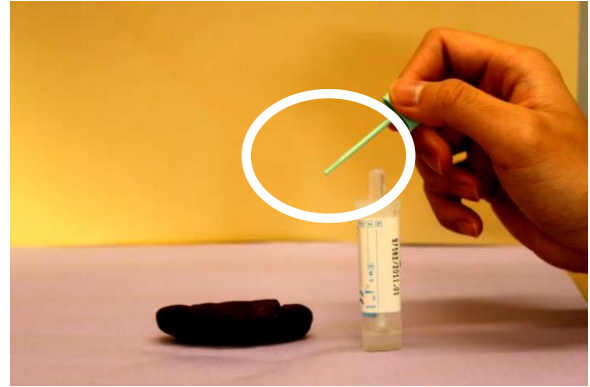

3. 用試管棒底部坑紋插入大便 6 個不同地方，以收集樣本。
Scrap the stick on the stool in 6 different areas until the grooved part is covered by the samples.



4. 用力將試管棒插回樽內，並扭動關上。於空白位置填上收集大便日期和時間。
Insert the stick and twist the cap closed. Write down the date and time on the surface of the bottle.

